

## Care First Aid PLUS AidTick-Out To AidTick-Out Ticks-2-Go

Ticks live in gardens, dunes, meadows and woodland. Have you been in the countryside? Check yourself for tick bites!

If you have a tick on your skin, ensure that the tick is removed within eight hours to minimise the risk of infection. Do not anesthetize the tick with alcohol or other antiseptics and do not heat the tick with a cigarette or other hot objects. If the tick is stunned or heated (with a cigarette), it goes into shock and responds by releasing pathogens into the body through its saliva. A good tick remover will help prevent the tick from passing on harmful bacteria in its saliva.

Once the tick has been removed you can disinfect the area with alcohol. Make sure to write down the date of the tick bite, the site of the bite on your body, and the area where you contracted the tick bite.

A tick can be easily removed using the Care Plus® Tick-Out Ticks-2-Go. This exclusively designed tick remover has two different notches for removing ticks. Take hold of the tick immediately behind its head and dig it out. With the Care Plus® Tick-Out Ticks-2-Go, you can even remove the most obstinate tick in its entirety - that is, including its head and legs - without difficulty. This tick remover does not exert any pressure on the tick, so the days of squeezing or 'tearing off' ticks are a thing of the past. The Care Plus® Tick-Out Ticks-2-Go can simply be attached to your keyring, so it is always within easy reach.



Article number

38396

EAN code

8714024383965

Weight

25 g

Dimensions packaging Ixwxh

30 x 80 x 135 mm

**Order quantity** 



## **IN SHORT**

- · Removes ticks quickly and safely with just a simple sliding motion
- · Specially developed instrument with two notches for dragging or digging out ticks
- · Prevents the ticks being squeezed or 'torn off'
- · Removes every type of tick, large and small